## Matelas gaufrier APLOT

## CONSEILS

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## APLOT waffle mattress

## TIPS

The best way to prevent pressure ulcers in the lying down position is to change the contact points of the body on the support on average every three hours to limit compressing vessels for too long around bony prominences and to allow tissues to be irrigated.

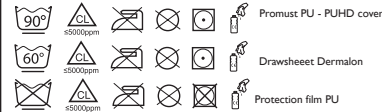
## INDICATIONS

- For patients with low to average risk of pressure ulcers, evaluated according to a tested scale and clinical judgement.
- Established pressure ulcers: follow medical recommendations.

## INSTALLATION

• Position the mattress so that the label sewn into the cover is situated at the head end. Position the mattress so that the blocks are uppermost

## CLEANING / DISINFECTION

Item treated with a biocidal substance that is safe for the user. [www.winnicare.fr](http://www.winnicare.fr)

## SAFETY

- Limit it to users weighing between 40 and 120 kg.
- **Precautions for protective coverings**
- Systematically protect the (underlay) mattress by means of a protective covering which is always delivered dry: if the foam has repeated contact with fluid, its properties will be modified regarding supporting the body uniformly and constantly as well as helping to lower pressure values.
- Prohibit scouring, stripping or dissolving products and sharp objects in direct contact with the protective covering.
- Make sure you regularly inspect the normal wear on the protective covering and replace it if it is damaged (holes, cracks, changes in the surface, transparent when looked at in the light).

## Precautions for materials

- Avoid prolonged damp contact with the foam.
- Perform regular visual checks as to the state of the foam: the presence of visible collapse of blocks or tearing to the base are signs of product aging which compromises the support properties which aid the prevention of ulcers.

## STORAGE

- APLOT supports should preferably be stored flat. Avoid exposing the foam to direct sunlight or humidity.

## DISPOSAL

- Do not burn or dump the support and its protective covering outside areas intended for this purpose.

## GUARANTEE

The duration of the guarantee for an APLOT mattress is 2 years. This guarantee starts on the date of purchasing the product from your distributor. In the event of single patient usage, the guarantee covers the support and its protective covering. In the case of multi-patient usage, we recommend replacing the protective Promust covers every 2 years. Dermalon protection is advised for single-patient usage. This guarantee does not cover normal wear on the product and its protective covering and does not substitute legal guarantees. Please contact YOUR DISTRIBUTOR and show him your faulty product. They will carry out the necessary procedures with our company in order to carry out repairs or make an exchange.

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- The estimated lifetime of EPSUS mattress is 5 years.

## COMPOSITION OF PRODUCT

- The mattress is 100% High Resilience (HR) moulded polyurethane foam.

## SUPPLEMENTARY TECHNICAL INFORMATION

Maximum surface pressures (mm Hg) for indicated patient weight. Measured on a flat (0° tilt) mattress

Reference	Max.Weight	Head	Pelvic	Heel
Mattress + Promust PU Cover	120 kg	89	72	82

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