

**Mangar**  
INTERNATIONAL



Safe  
*relaxing*  
bathing

*Quality bathing solutions for everyday independence*

# *Practical* and affordable products designed especially for you

Ever since we invented the world's first portable powered bath lift in 1981, we have worked hard to maintain our position as one of the world's leading providers of bathing solutions.

Our award-winning products are carefully designed to get you enjoying the sensation of bathing again. We take pride in producing easy to use and often life changing products that will help you remain safe and independent in your own home.

The Bathing Cushion is just one of those life changing products. An ingenious inflatable and fully portable bath lift you can store away neatly after use.

If, however, you require a little more support than our Bathing Cushion provides, our Archimedes Bath Lift will allow you to relax while taking the weight off your joints.

If bathing is becoming increasingly more difficult for you, please read on to discover how you really can enjoy safe, easy and worry free bathing, all at the touch of a button.

With best wishes  
*for more enjoyable bathing*

Andrew Barker, Managing Director



lightweight  
and  
portable

# The Bathing Cushion

When getting in and out of the bath becomes a problem, try the Bathing Cushion and you'll soon rediscover all the pleasures of a soothing warm bath. It gently lowers you down and raises you back up again at the touch of a button and is one of the most easy to use bath lifts on the market.

## With the Bathing Cushion you benefit from all this and more

- ✓ You can fit and remove your Bathing Cushion in seconds!
- ✓ Get in and out of your bath with total confidence
- ✓ Lie down completely so your entire body is covered with water just like a proper bath
- ✓ Leave some air in the backrest for greater levels of comfort
- ✓ It's lightweight, stable and easily portable, so you can take it with you, wherever you go!
- ✓ The clever design adapts to the size and shape of your bath, so there are no expensive bathroom alterations
- ✓ Being battery operated, there are no dangerous power cables, like other products on the market
- ✓ You stay in control with the fully waterproof hand control
- ✓ You stay safe as you will only be lowered into the bath if there is enough battery power to raise you back up again
- ✓ Lifts up to 150kg (24 stone)

## Fully portable, versatile and fuss free

Whether in your own home, away from home, or staying with friends or family, we'll show you just how user-friendly and versatile the Bathing Cushion is!



# The Bathing Cushion does all the work for you

*It really is as easy as 1, 2, 3...*

## **1 Get in the bath with ease**

Sit comfortably on your already inflated cushion. Deflate down gently.



## **2 Lie back and relax**

Your entire upper body is covered with water. Just like a proper bath.



## What our customers say...

*"...one of the most helpful aids I have – I feel completely safe and comfortable and it is a real luxury to be able to soak in a bath of hot water for the first time for several years. Thank you."*

**Mrs T, Devon**

*"Now I have the Bathing Cushion I can get in and out of the bath without help, giving me the invaluable advantage of living alone and independently. A very sincere thank you, it has opened up a whole new world to me."*

**Mrs S, Worcestershire**

With the Bathing Cushion you enjoy a blissful bathtime, every time!

### 3 Get out with confidence

Inflate your cushion with the fingertip control. Raise up slowly, then safely out.



*“They say miracles seldom happen but for me the Mangar Bathing Cushion is one. Increasingly crippled by arthritis, I found it agony to climb into my ancient, narrow bath but now I can sit comfortably back and be lifted into the bath and, when I am ready, lifted out again. It does all the work for me in blissful comfort.”*

**Mrs Jackson, Llandrindod Wells**

## FAQs

### **Q. How do I clean the Bathing Cushion?**

We would recommend that you rinse the Bathing Cushion with clean water after each bath and leave it to dry inflated. To ensure maximum adhesiveness we would further recommend that you remove the Bathing Cushion from the bath at least once a week and clean the bottom of the suction cups as well as the surface of the bath.

### **Q. How many baths can I get out of one charge?**

This depends on a number of factors; how much you weigh, how old the batteries are and how well they have been maintained and charged since purchase. We recommend you leave batteries on charge when not in use. This means batteries will provide enough power for several baths and will be in optimum condition for a longer life.

### **Q. Am I eligible for VAT relief?**

If you are chronically sick or disabled you may be able to claim VAT relief on the Bathing Cushion. VAT relief is controlled by HM Revenue and Customs (not Mangar). Guidance on eligibility is available from the National Advice Service on 0845 0109000. We can provide you with a VAT relief form but it is your responsibility to ensure that you qualify.

### **Q. Can the Bathing Cushion be overinflated?**

No, the Bathing Cushion is fitted with a pressure relief valve which prevents this from happening. A further safety feature on the Bathing Cushion means that you can also only be lowered into the bath if there is enough battery power to raise you back up again.



lightweight  
and  
battery  
powered

## The Archimedes Bath Lift

If you need a little more support, this lightweight, battery-powered bath lift offers all the comfort of a high backrest and long seat while still maximising your leg room. So you get to relax while taking the weight off your joints.

- ✓ Great alternative to a walk-in bath
- ✓ Comfortable and reassuring fixed seat
- ✓ Simple to fit or remove – separates into three manageable pieces
- ✓ Lightweight – the heaviest part weighs just 5.2kgs
- ✓ Battery operated – no dangerous power cables for you to worry about
- ✓ Fully waterproof hand control
- ✓ Safety feature: will only lower you into the bath when there is sufficient battery power to raise you back out again
- ✓ Lowers to within 7cm of the bottom of the bath
- ✓ Handy battery level indicator light
- ✓ Unique sucker release system designed to avoid having to release suckers one by one
- ✓ Lifts up to 150kg (24 stone)
- ✓ Simple to use controls
- ✓ Easy to clean

The best way to decide on a bathing product is to discuss your specific needs with one of our fully trained experts. They can talk you through the features and benefits of the Bathing Cushion and the Archimedes and guide you on which will be more suitable for you.



*We're dedicated*  
to helping you...

Over  
300,000  
bath lifts  
sold

Our highly trained, experienced and discreet personnel are happy to discuss your individual requirement and answer all your questions about the Bathing Cushion or Archimedes Bath Lift. They will take your order only when you are confident that our products are right for you.



Product Warranty  
*Your peace of mind  
is our peace of mind.*

We strive to provide all of our customers with reliable and durable products of the highest quality and each comes with a standard Mangar guarantee, for added peace of mind.



**Mangar**  
INTERNATIONAL

# You'll make the right choice with Mangar International

As a careful customer, you're right to do your research. Here are just a few reasons why you should choose us.

- ✓ We are a trusted company and a member of the British Healthcare Trades Association (BHTA), operating to the Trading Standards Institute (TSI) code of practice
- ✓ We are an award-winning company in the global healthcare market, renowned for the quality and durability of our products
- ✓ 30 years experience of producing the highest quality bathing products that ensure you maintain your independence as well as your dignity
- ✓ Manufactured in the UK, our products are expertly designed to fit in with your everyday life and require no permanent fixtures or alterations to your home
- ✓ If you decide to move home, go on holiday, or simply want to prepare the bath for another person, our products can be removed or re-fitted in a matter of seconds
- ✓ All our new products and developments are subject to trials and assessments by end users and healthcare professionals for added peace of mind
- ✓ We carry out stringent quality assurance tests through every stage of production to ensure our products meet our customers' high expectations
- ✓ We strive to produce better life-changing independent living solutions, so you can enjoy a better life, every day!

## Contact us today

Freephone 0800 2800 485

[sales@mangar.co.uk](mailto:sales@mangar.co.uk)

[www.mangar.co.uk](http://www.mangar.co.uk)

## Mangar International Ltd.

Presteigne, Powys

Wales, LD8 2UF, UK

**Mangar**  
INTERNATIONAL