




This Quick Start Guide is not intended to replace the full user instructions. Please read the full user instructions before using the Ekamove.





Set Up

1. Strap the Ekamove to the bed frame with sensors level with patients shoulders (On profiling beds fit straps to moving parts of frame only).
2. Fit the control unit to the foot of the bed and clamp in place.
3. Run sensor lead and air hoses to the control unit ensuring that they avoid any pinching or trapping points and connect to underside of unit.

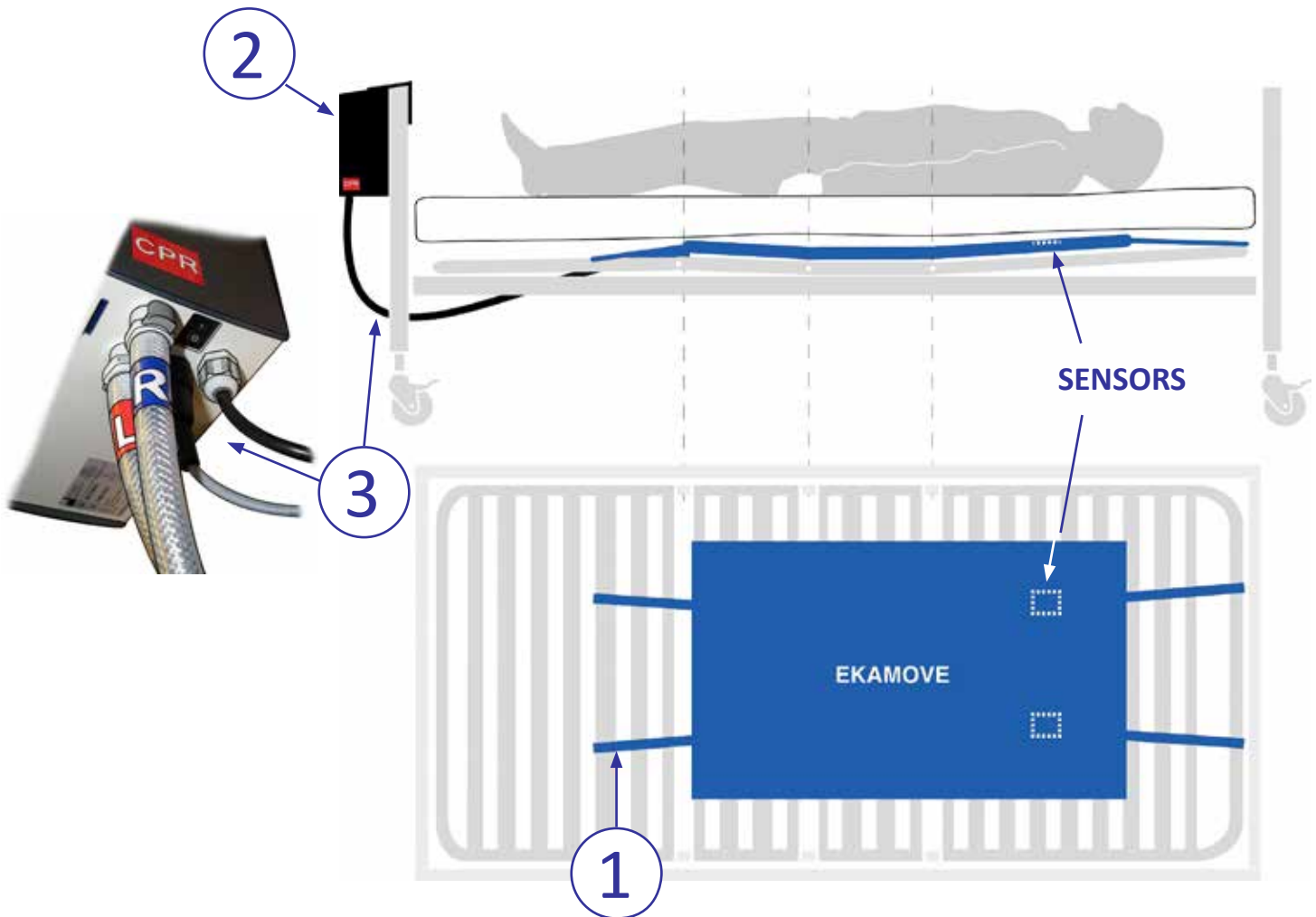

Programming

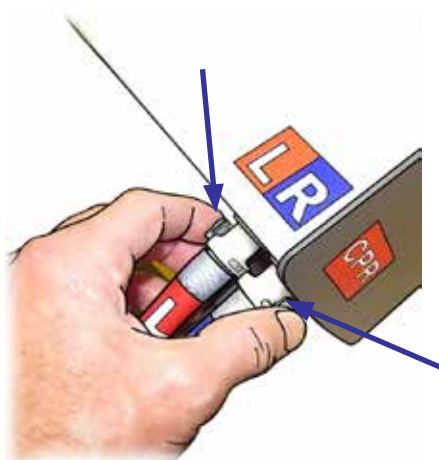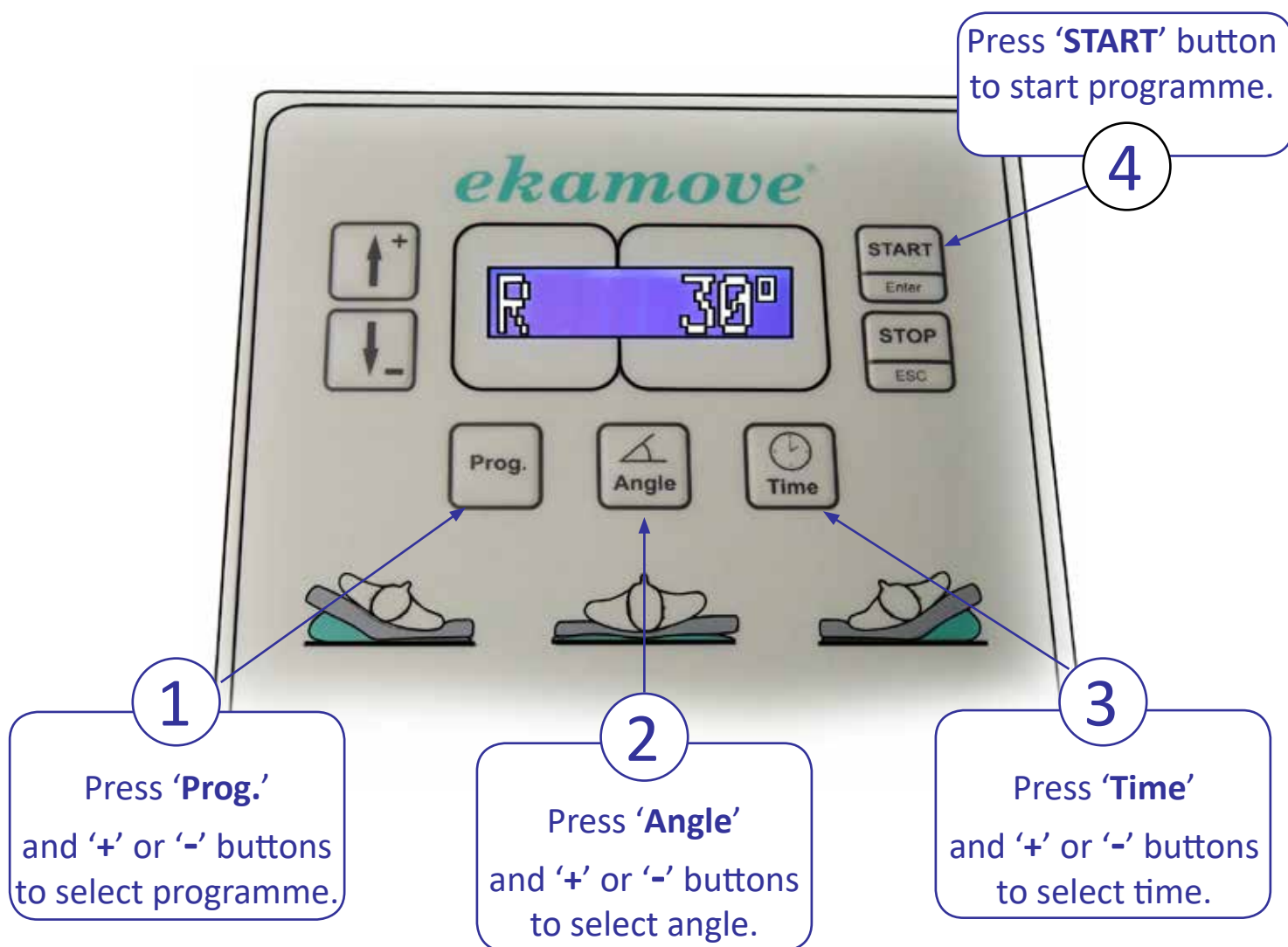

NOTE: The air hoses can be released to vent air quickly by pressing together the two grey lugs where they connect to the control unit.

Mangar Health

Presteigne, Powys, Wales, LD8 2UF, UK

Tel: +44(0)1544 267674

Email: sales@mangarhealth.com

Web: www.mangarhealth.com