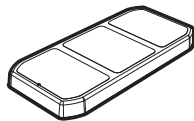
## ASKLÉSANTÉ

## Matelas simple MORFÉA

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
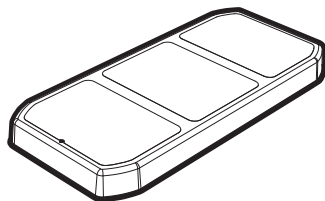
## MORFÉA Mattress

## ASKLÉSANTÉ'S ADVICE :

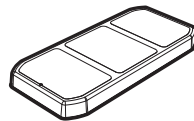The first line in bed sore prevention involves changing the pressure points of the body on the mattress an average of every two to three hours, to limit prolonged compression of blood vessels in relation to bony protrusions and to allow tissue irrigation.

Guarantee: 2 years

**Risk level :** none to low

**Max weight:** 120 kg

**Positioning:** 

**Specifications:** High Resilience Polyurethane Moulded Foam

**Torso elevation :** 45 °

**Dimensions :** 1980 x 893 x 145 mm

**Density:** 35 kg/m<sup>3</sup>

## INDICATIONS:

Comfort during bed rest. The patient is self-mobile and presents only a weak risk of ulceration. Discontinuous bed rest situation.

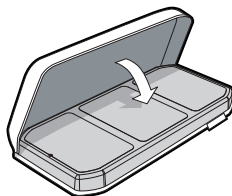
## CONTRAINDICATIONS :

Bed rest for longer than 10 hours where the patient finds it difficult to be self-mobile. Ulcers form at pressure points

## WARNINGS :

Multi-patient usage only where a visual check can be performed between each patient with regards to integral protection. The application of chlorinated products (≤ 1000 ppm for everyday cleaning - bleach solution ≤ 5000 ppm for extreme situations). Any break in the surface of the cover requires the cover be replaced

## INSTALLATION:

- Open the cover by pulling on one of the two 3-sided fastening sliders.
- Place the cover on the bed base, ensuring that the large label is situated on the lower part.
- Place the mattress, widest part, in contact with the bed base.
- Close the cover; the flap will cover the sliding opening.

## CLEANING / DISINFECTIION

• **VHMB14 :** Cover PVC - without Phtalates - **Legal Guarantee :** 2 years

• **VHIPH/MPC :** Cover PUHD - **Legal Guarantee :** 2 years

• **VHIP2/MPC :** Cover Protection PU - **Legal Guarantee :** 2 years

Item treated with a biocidal substance that is safe for the user. [www.winnicare.fr](http://www.winnicare.fr)

## GUARANTEE :

The guarantee is valid against all consequences of faults or hidden flaws resulting from the design, structure or materials not detectable, which may cause direct or indirect harm. The guarantee does not cover modifications to the materials or structures resulting from inappropriate use, care or storage of the mattress, according to the recommendations of ASKLÉSANTÉ. For all claims, please contact your service provider returning the products concerned with its validated guarantee certificate (stamped and dated at purchase) and quote the lot number. The mattress guarantee period is two years .

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