

COVID-19 Cleaning your Lifting Cushion

When responding to a fall, Mangar recommend the follow steps with our products every time you use the Camel or Elk Lifting Cushion;

1. **Wash your hands and put on disposable gloves**
(Make sure to not touch anything other than the products or patient during this time)



2. **Collect the Camel or Elk Lifting cushion**



3. **Attend to the fallen patient using the lifting cushion as per your training on the product**



4. **Once patient is off the lifting cushion, and before you deflate, thoroughly disinfect the cushion**



5. **Pack the cushion away as normal and return to storage location**



6. **Throw away disposable gloves and wash your hands**

