

Mangar
HEALTH



**EVERY NIGHT IS
BATHTUB NIGHT
WITH A MANGAR BATHING CUSHION**

Enjoy the benefits of a long soak, independently

When getting in and out of the bath becomes a problem try the Mangar Bathing Cushion and you'll soon rediscover all the pleasures of a soothing warm bathtub. It gently lowers you down and raises you back up again at the touch of a button.

- Lie down completely so your entire body is covered with water
- Fit and remove your Bathing Cushion in seconds - no costly bathroom alterations required
- Get in and out of your bathtub with total confidence
- Leave some air in the backrest for greater levels of comfort
- It's lightweight, stable and easily portable, so you can take it with you wherever you go
- The clever design adapts to the size and shape of your bath, so there are no expensive bathroom alterations
- Being battery operated, there are no dangerous power cables
- You stay in control with a fully waterproof hand control
- Lifts up to 150 kg (24 stone)


“ My bathing cushion arrived last week and I've now had my first bath in more than 2 months. I LOVE MY MANGAR BATHING CUSHION!!!!!!!!!!!!!! My hubby helped me the first few nights, just until I got confident and I am now having baths whenever I want! I thank you from the bottom of my heart-er bathtub!”

Lorna Young, British Columbia

Easy and safe to use


1 Get in the bathtub with ease

Sit comfortably on your already inflated bathing cushion. Use the handset to lower you down gently.

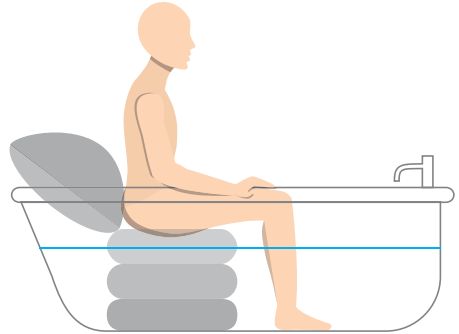


2 Lie back and relax

Your entire upper body is covered in water – just like a proper bath.

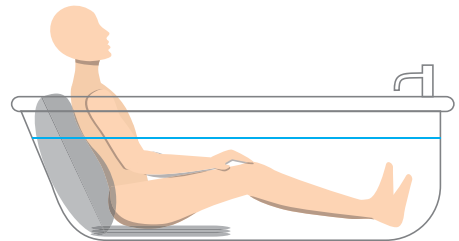

3 Get out with confidence

Reinflate your bathing cushion and step out safely. Have the peace of mind that you will only be lowered into the bathtub if there is enough battery power to raise you back up again.

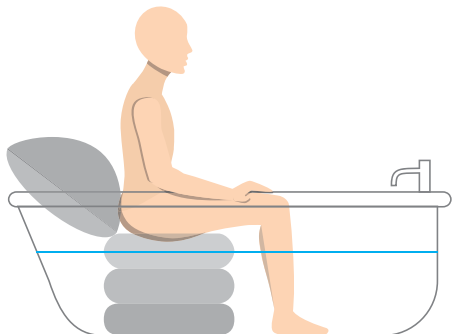

Over 300,000 sold worldwide

The Bathing Cushion is one of those life-changing products you'll wonder how you ever lived without. An ingenious, inflatable and fully portable bathtub lift, it can be stored neatly away when not in use.

If you have any questions about the Bathing Cushion you can talk to our experienced and discreet customer care team. If you have any doubts, don't worry. Our returns policy means you can try it dry in your bathtub first to make sure it's the right product for you.

As inventors of the first ever powered bathtub lift we know our products are reliable and durable. Manufactured in the UK, the Bathing Cushion comes with a standard one year guarantee. For additional peace of mind ask us about our servicing and extended warranty offers.

All Mangar products are shipped from Rhode Island, USA.

Contact us today

855-9-MANGAR*
canada@mangarhealth.co.uk

www.mangarhealth.com

